


LTV[®] 1200 EMERGENCY SETUP CARD


FOR 24 HOUR
ASSISTANCE
1-800-754-1914




STEP 1 CHOOSE A POWER SOURCE

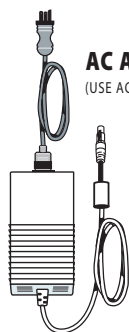

AC ADAPTER
(USE AC POWER WHEN AVAILABLE)


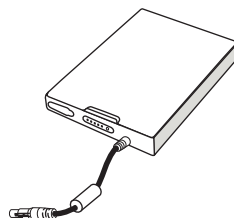


LI ION BATTERY

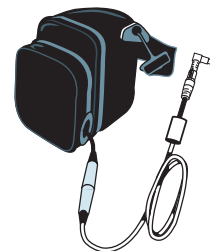


LEAD ACID BATTERY

STEP 2 CONNECT LTV TO POWER SOURCE

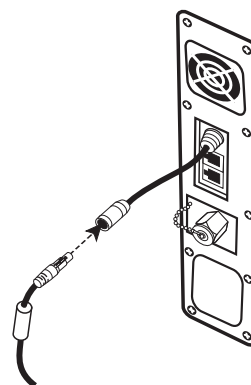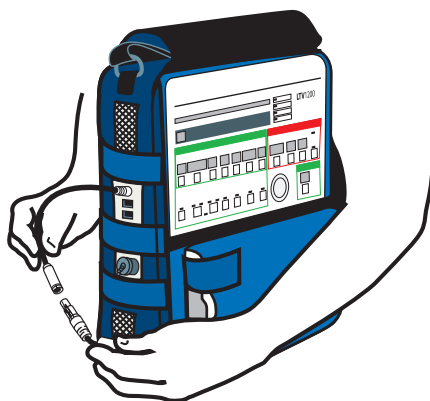


STEP 3 CHOOSE A CIRCUIT

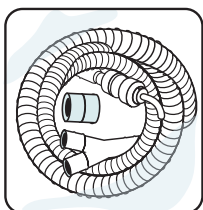


**OPEN BAG
REMOVE PATIENT
CIRCUIT AND
22 mm ADAPTER
FROM BAG**


HUMIDIFIER CIRCUIT/ OPTIONAL

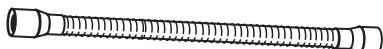

PATIENT CIRCUIT ADULT
>20 kg
>44 lbs

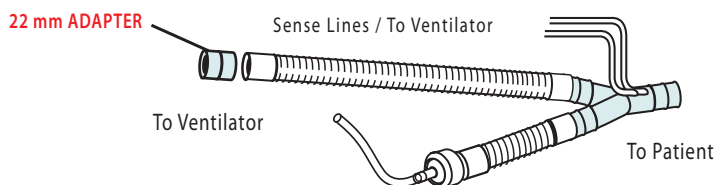




PATIENT CIRCUIT PEDIATRIC
≤20 kg
≤44 lbs

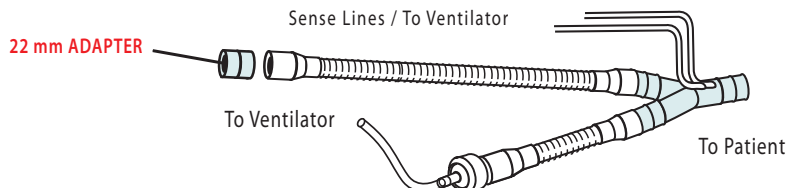

STEP 4 CONNECT CIRCUIT TO LTV




STEP **A** ATTACH 22mm ADAPTER
(FOUND IN CIRCUIT BAG)

STEP **B** CONNECT CIRCUIT TO
22mm ADAPTER

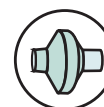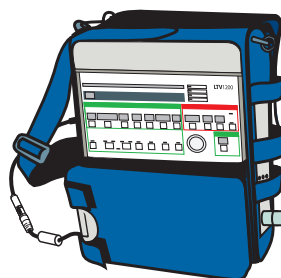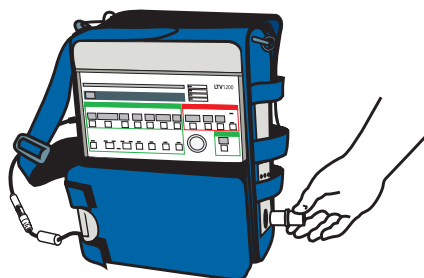


FILTER OPTIONAL



STEP 5 CONNECT SENSE LINES TO LTV

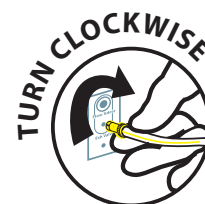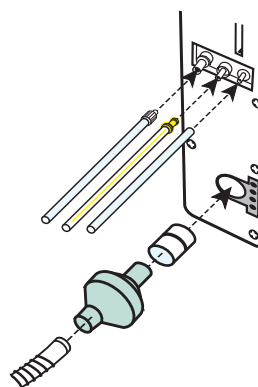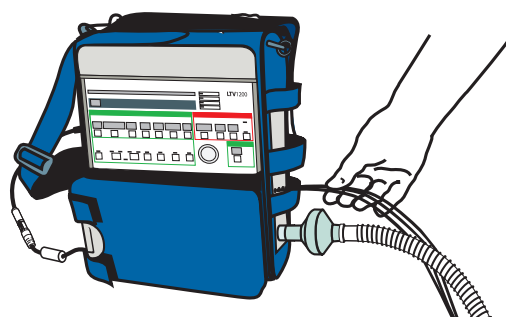

STEP 6 TURN LTV ON

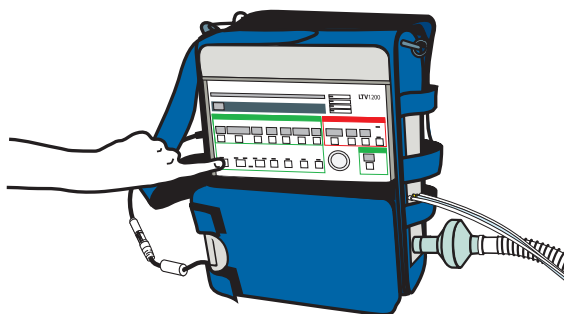


On/Standby

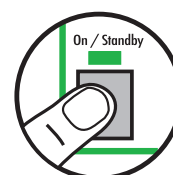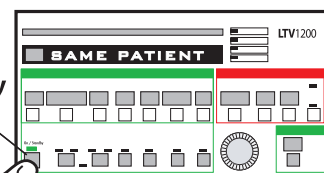


STEP 7 PATIENT SETTING

ONCE POWER IS ON SEE
SAME PATIENT IN DISPLAY

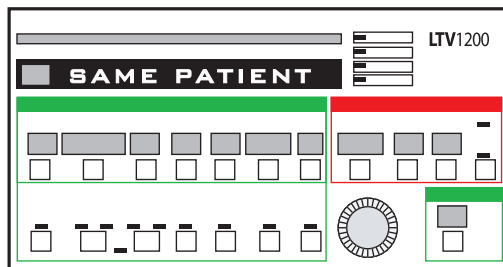


STEP 8 NEW PATIENT SETTING

STEP A ROTATE SET VALUE KNOB
CLOCKWISE UNTIL
NEW PATIENT IS DISPLAYED

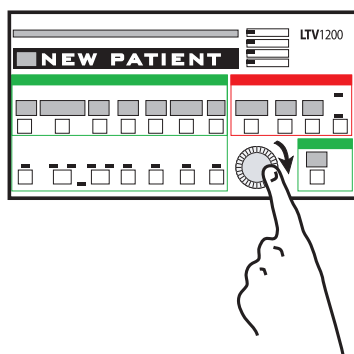


STEP B PRESS SELECT WHEN
NEW PATIENT
IS DISPLAYED

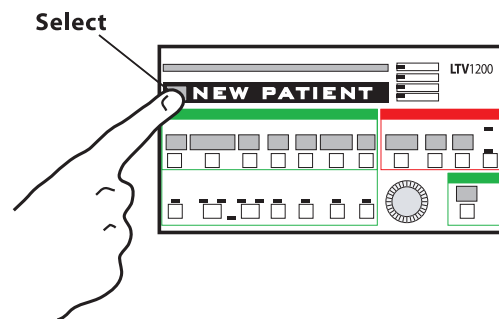


STEP 9 SET FOR PATIENT SIZE

STEP A ROTATE SET VALUE KNOB TO CORRECT
PATIENT SIZE. (INFANT,PEDIATRIC, OR ADULT)

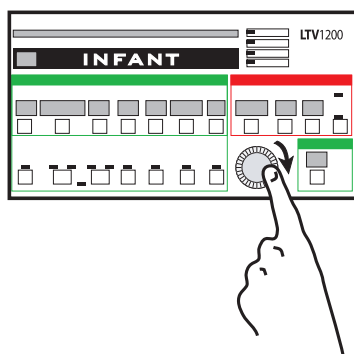



STEP B PRESS SELECT WHEN PROPER PATIENT
SIZE IS DISPLAYED

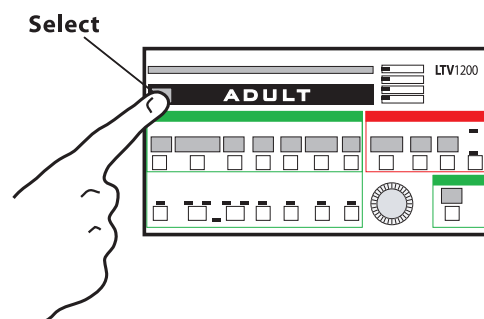

? MISTAKE


If you make a mistake, turn the LTV off by holding the
ON/STANDBY button for 3 seconds. Then return to Step 6.

STEP 10 CONNECT CIRCUIT TO ET-TUBE OR MASK

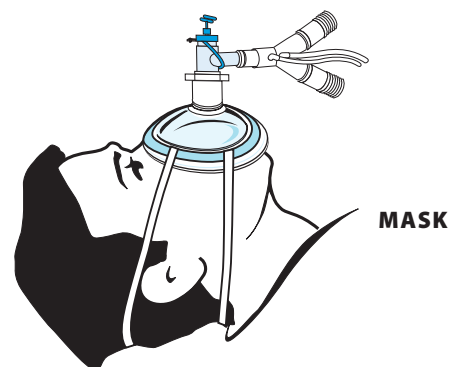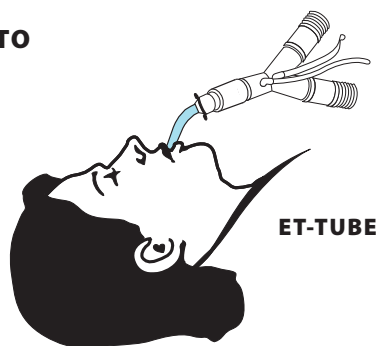

NOTE ! ALARM

NOTE: THE LTV® WILL ALARM UNTIL IT IS CONNECTED TO THE PATIENT. YOU MAY PRESS THE SILENCE/RESET BUTTON.

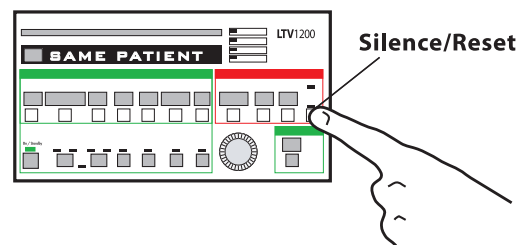

STEP 11 OPTIONAL//USING A MASK

CHOOSE A MASK (PEDIATRIC OR ADULT)




PEDIATRIC




ADULT

CONNECT ELBOW TO MASK

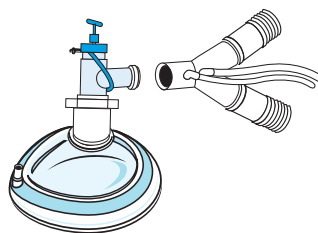


CONNECT TO PATIENT

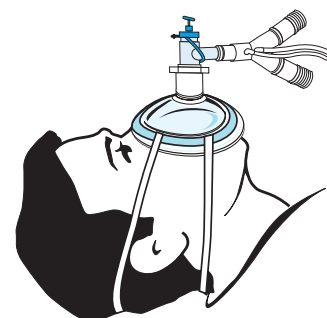

STEP 12 OPTIONAL // USING AN HME AND/OR FLEX TUBE AS SHOWN FOR PATIENT COMFORT

FLEX TUBE

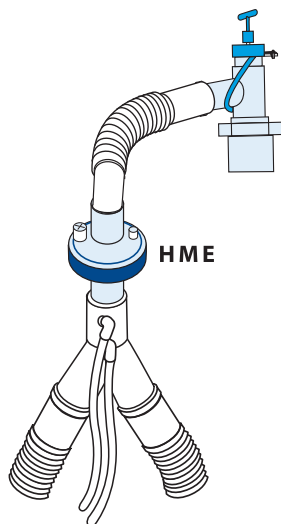

THIS EMERGENCY SETUP CARD IS INTENDED TO PROVIDE EMERGENCY INSTRUCTIONS FOR INITIAL SETUP OF THE LTV 1200 VENTILATOR. THE INITIAL SETTINGS MAY NEED TO BE ADJUSTED BASED ON THE PATIENT'S CLINICAL CONDITION. FOR COMPLETE INFORMATION REGARDING SETUP AND OPERATION OF THE VENTILATOR REFER TO THE LTV 1200 OPERATOR'S MANUAL



Cardinal Health
Respiratory Care
17400 Medina Road, Suite 100
Minneapolis, MN 55447
Toll Free 800-754-1914
cardinalhealth.com/viasys